

# PREVENTIVE MAINTENANCE CHECKLIST SIX (6) MONTH

SERIAL NO. \_\_\_\_\_ DATE \_\_\_\_\_  
 MACHINE I.D. NO. \_\_\_\_\_ MACHINE HOURS \_\_\_\_\_  
 TECHNICIAN(S) \_\_\_\_\_ LABOR HOURS \_\_\_\_\_

See the Referenced Section of the 2008T Preventive Maintenance Procedures (part number 508033 Rev. R) for detailed instructions to perform the following:

<u>Reference Section</u>	<u>Procedure</u>	<u>Value</u>	<u>Completed By:</u>
2.1	Clean Filters..... (Replace If Necessary)		_____
2.2	Inspect Pre-UF Pump Filter ..... (Replace If Necessary)		_____
2.3	Inspect UF Pump Check Valves..... (Replace If Necessary)		_____
2.4	Check High Voltage AC Connections..... (<0.2 ohms)	_____	_____
2.5	Calibrate UF Pump ..... (Range: 0.90 to 1.10ml)	_____	_____
2.6	Check Conductivity Calibration ..... (Range: $\Delta \pm 0.1\text{mS/cm}$ )	Machine Display	External Meter
		(Range: $\Delta \pm 0.2\text{mS/cm}$ )	_____
		TCD	External Meter
2.7	Check Temperature Calibration..... (Range for 37°C: $\Delta \pm 0.3^\circ\text{C}$ )	Machine Display	External Meter
		(Range for 39°C: $\Delta \pm 0.3^\circ\text{C}$ )	_____
		Machine Display	External Meter
2.8	Check Alarm Operation and Pressure Holding Test (N/A for 2008T BlueStar Premium)		_____
2.9	Verify pH (N/A for 2008T BlueStar Premium) ..... (Range 6.9 to 7.6)	_____	_____
2.10	Check Power Failure Alarm (N/A for 2008T BlueStar Premium) .....		_____
2.11	Final Checks .....		_____

**NOTES:**