

- NOTES:
- REFER TO FIGURE'S 1 - 4 FOR WIRING OF FILL VALVE, TRANSFER VALVE, MIX MOTOR, AND RECIRC. PUMP.
 - ATTACH STRAIN RELIEF P/N 368016-05 AND NUT 332020-03 TO FILL VALVE CABLE (P/N 160085) BEFORE ATTACHING FILL VALVE CONNECTOR.
 - CONNECT MIX MOTOR CABLE TO THE TWO TABS IN THE MOTOR HOUSING.
 - REMOVE AND DISCARD FACTORY INSTALLED WIRING FROM RECIRC. PUMP HOUSING. CONNECT THE RECIRC. PUMP CABLE TO THE TWO .250" TABS IN THE RECIRC. PUMP HOUSING.
 - THIS DIAGRAM SHALL BE USED FOR EITHER THE DISSOLUTION UNIT ASSEMBLY, P/N G97-283-04, AND HYDRAULIC ASSY, DISSOLUTION UNIT, P/N G97-283-02 OR 132 DRY ACID DISSOLUTION UNIT, ASSY, P/N 160138.

REV.	RELEASE DATE	EFFECTIVE DATE	ORIGINATOR	CN#	DESCRIPTION OF CHANGE	APPROVAL
A	10/20/06	10/20/06	D.BRIDGEDALE	19391	RELEASE	D.BRIDGEDALE
B	7/8/08	7/8/08	J.ESTRADA	25506	PART NUMBER CHANGE	J.ESTRADA
C	4/18/11	4/18/11	F. JARUMAY	35169	ADD ALT. SWITCH 363029-09	F.JARUMAY
D	10/07/13	10/12/13	F. JARUMAY	CN047217	INCORPORATE DRY ACID CONFIGURATION	B. HOLMQUIST
E	06/03/14	06/05/14	H. GARRIDO	CN049833	UPDATE NOTES. ADD PN 160142 TO CONTROL CONSOLE.	L. PANTOJA

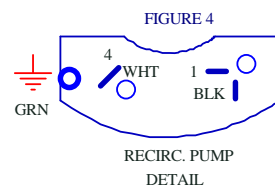
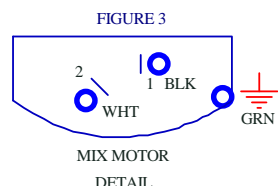
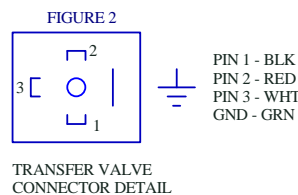
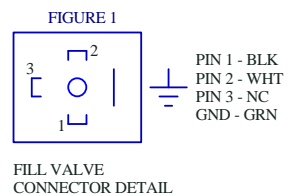


FIGURE 5 - FOR USE WITH THE 132 DRY ACID DISSOLUTION UNIT, ASSY P/N 160138



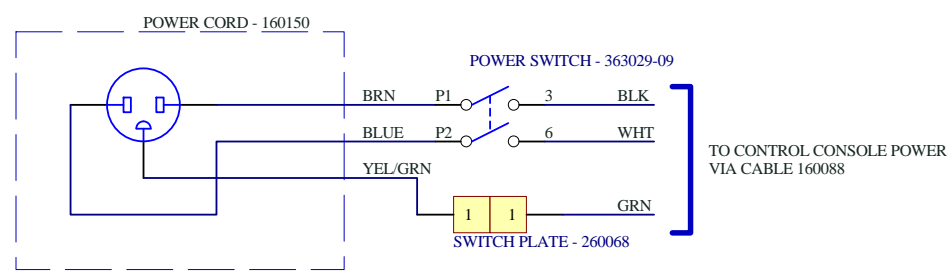


Figure 6 - FOR USE WITH THE HYDRAULIC ASSY, DISSOLUTION UNIT, P/N G97-283-04

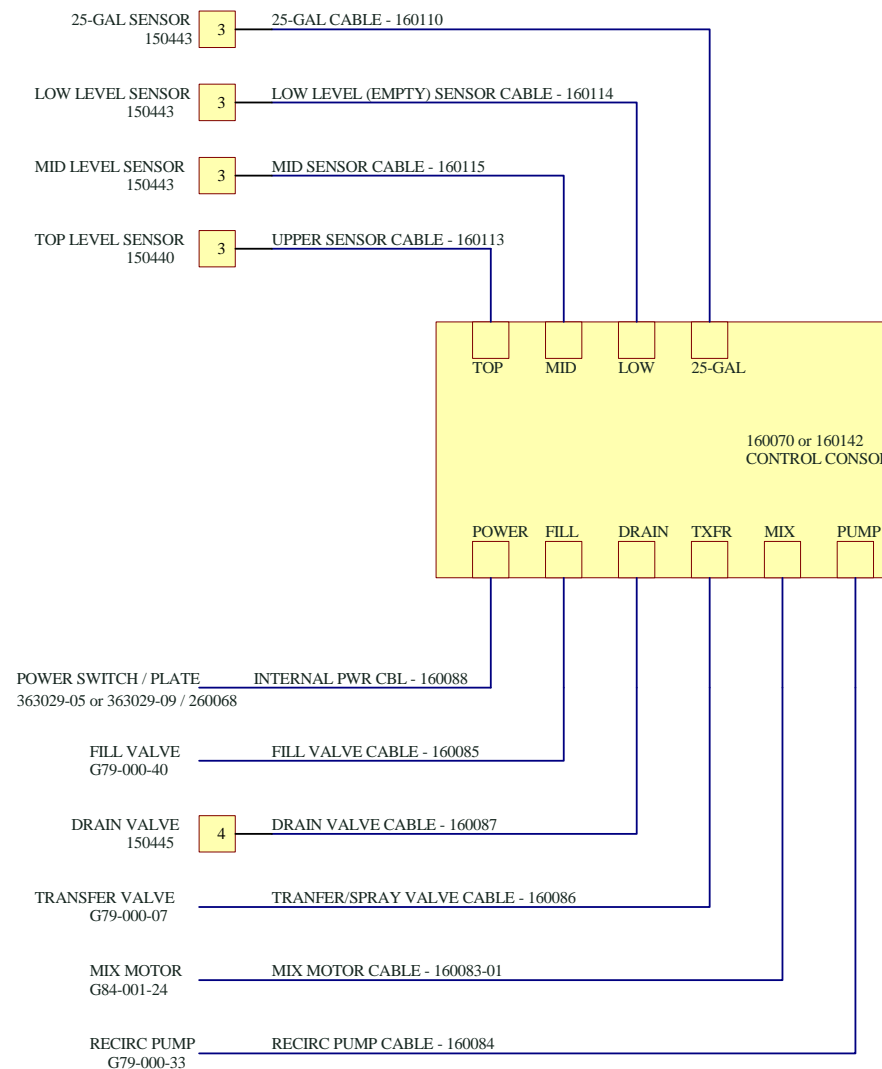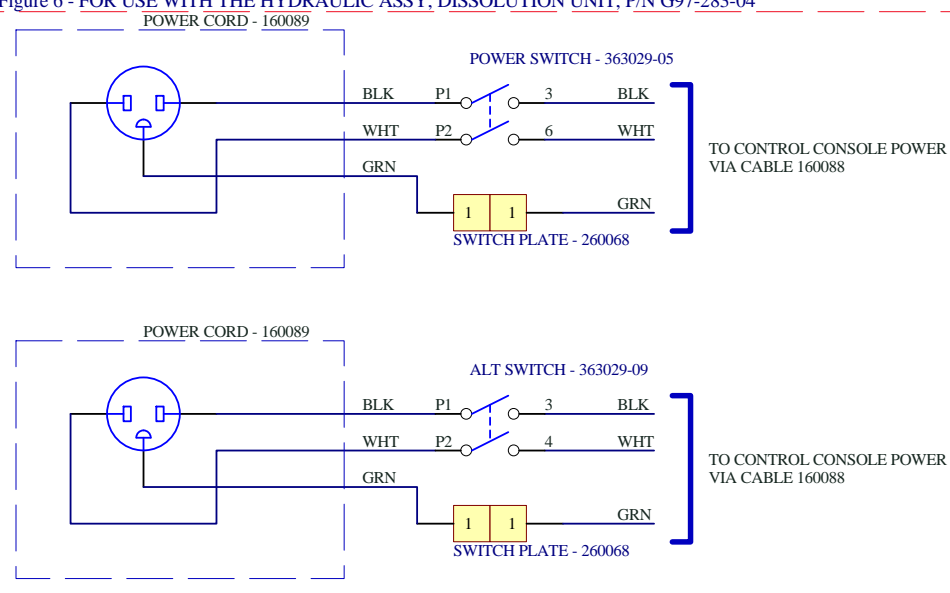

This document is the property of Fresenius USA, Inc. d/b/a Fresenius Medical Care North America and may contain confidential and trade secret information. This document may only be transferred as authorized by Fresenius Medical Care North America and then only by way of loan for limited purposes. It may not be reproduced in whole or in part and must be returned or destroyed upon request or upon completion of the purpose of the loan.

© Fresenius Medical Care North America 2006-2014

UNLESS OTHERWISE SPECIFIED		NAME	DATE
DIM ARE IN INCHES		DRAWN D.BRIDGEDALE	4/18/2011
TOL ON ANGLES ±0.5		DRAWN B.HOLMQUIST	10/20/06
2 PL 0.01, 3 PL 0.005			
INTERPRET DIM AND TOL PER ASME Y14.5-1994			
THIRD ANGLE PROJECTION			
MATERIAL		N/A	
FINISH		N/A	
DO NOT SCALE DRAWING		N/A	

Fresenius Medical Care

TITLE:
WIRING DIAGRAM, GRANUFLO / DRY ACID 132

SIZE **B** DWG. NO. **290316** REV. **E**

SCALE: 1:1 CAD: PROTEL SHEET 1 OF 1