



Fresenius Medical Care

Fresenius Medical Care North America Service Bulletin

Equipment: GranuFlo[®] Dissolution Tank - 132 Gallon
Bulletin: 11-GRN-002 Rev A
Subject: Propeller Shaft Assembly

Originated By (Dept.):	Reviewed By	Approved By	Effective Date
J. Fox (TSD)	Kenneth Cogswell	CN039913	11/14/2011

Distribution: Dir TS (B. La Bier), TS Mgr (K. Cogswell), Dir QS (M. Owen), QE (R. Fox), Eng (C. Chau), QC Mgr (D. McKenzie), Tng (J. Barr), Prod Mgr (J. Horne), Corp TS Mgr (C. Woods), Canada (Sean McFaul), Mexico (Jorge Naranjo)

1.0 PURPOSE

To inform all Field Service and Technical Support personnel of a new Propeller Shaft Assembly for the 132 gallon GranuFlo[®] Dissolution Tank.

2.0 SCOPE

A new assembly has been created for the 132 gallon GranuFlo[®] Dissolution Tank. It combines the two propellers and the propeller shaft into one assembly. The figure below shows the new assembly:

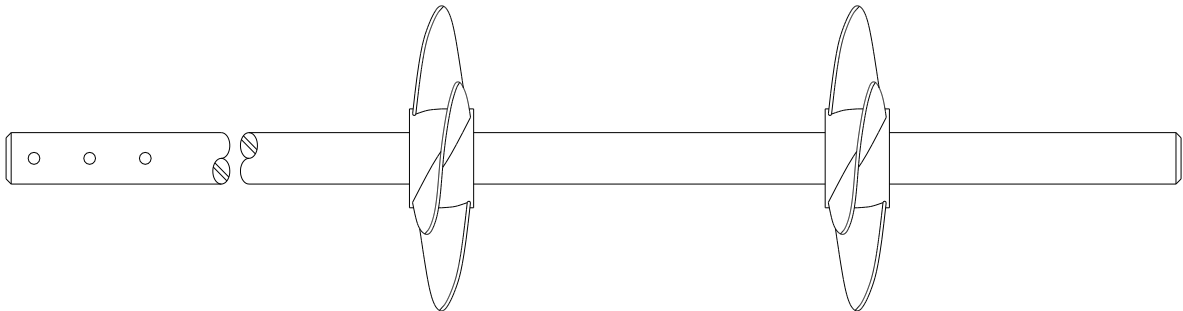


Figure 1 - New Propeller Shaft Assembly (P/N 160154)

Caution: Do not attempt to disassemble the propeller shaft assembly. It must be installed as an assembly due to a specific manufacturing process.

The propellers (P/N G84-001-42) and the propeller shaft (P/N 250452) are no longer available as separate spare parts.

If there are any questions regarding this bulletin, contact Fresenius Technical Support at 800-227-2572.

© 2011, Fresenius Medical Care North America, All rights reserved.