

## Fresenius Medical Care North America Service Bulletin


Equipment: All 2008 Series Hemodialysis Machines  
 Bulletin: 16-FRS-002 Rev A  
 Subject: New 6.3 Amp Power Supply Fuse

### 1.0 PURPOSE

To inform all Field Service and Technical Support personnel, as well as, Fresenius Medical Care-trained customers of an updated 6.3 Amp fuse used in the power supply for all 2008 Series Hemodialysis machines.

### 2.0 SCOPE

An updated version of the 6.3 Amp fuse used in positions SI2 and SI6 on the Power Control board has been created. Unlike the original fuse, which has a white body, the new fuse has a clear body. See Figure 1 for the fuse locations, the difference in appearance and the part number for the new fuse.

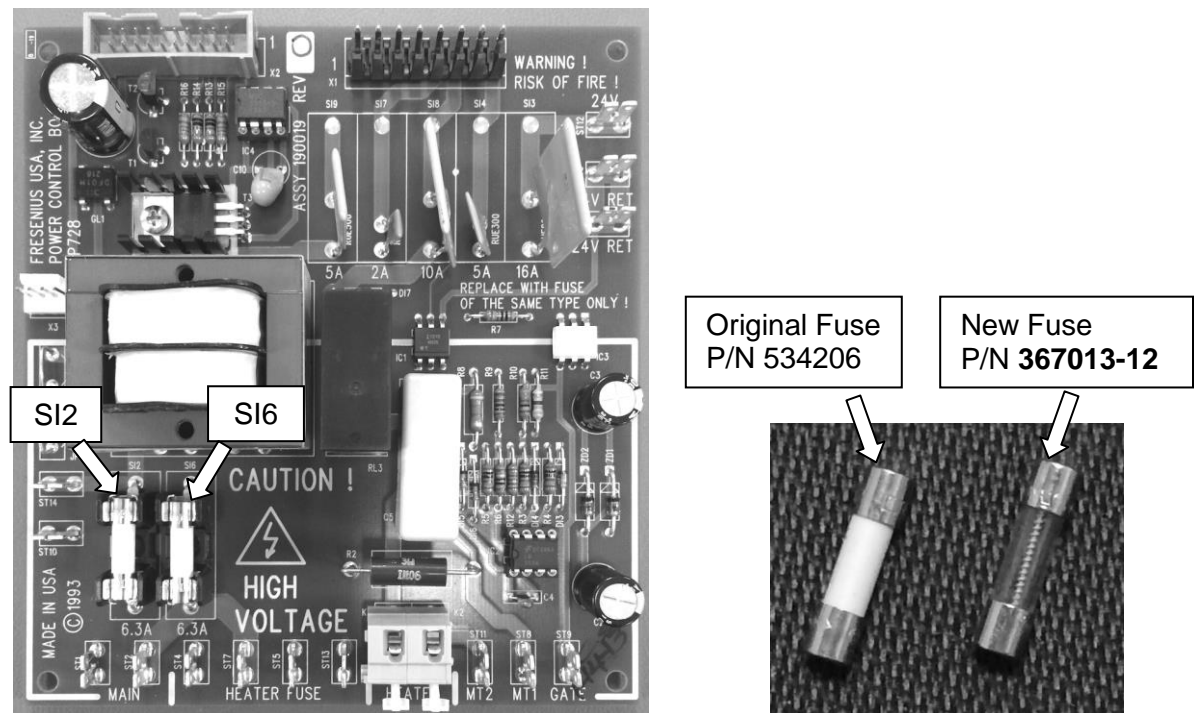

**Figure 1**

If there are any questions regarding this bulletin, contact Fresenius Medical Care Technical Support at 800-227-2572

© 2016, Fresenius Medical Care North America, All rights reserved.