

Simply better.

Reduced alarms and more personalized therapy options with **Liberty[®] Select 2.9**



Fresenius Medical Care is pleased to announce new software and hardware enhancements for the **Liberty Select Peritoneal Dialysis Cyclers**, promoting ease-of-use, patient satisfaction, and clinical excellence. Welcome to Liberty Select 2.9!



Less Cause for Alarms

Removal of the Scale

Promotes a quieter therapy experience for patients and less alarm monitoring for nurses! Note, the Liberty Select cycler will continue to measure solution volume in the heater bag throughout the treatment.



Fewer Sounds, Sounder Sleep

Optional Flow Alert Notification

The Liberty Select cycler notifies patients with a Flow Alert when slow flow occurs, often interrupting sleep. For a more individualized experience, patients can now work with their clinician to adjust the Flow Alert notification ON or OFF. If OFF is selected, patient will experience fewer alerts and sleep interruptions. Additionally, Flow Alert occurrences are now visible in the IQsystem[®] Alarm History, enabling up-to-date, data-driven therapy adjustments.



A More Personal Touch

Easier Data Entry and Monitoring

The "Patient Data" default screen will now display blank spaces for all values, making it easier for patients to enter in their data daily. As a clinician, you'll have better visibility of patient health data for timelier, more effective patient management.



Training Tools that Build Confidence

Gain Knowledge and Guidance to Help Patients

New, robust training resources and online learning modules include troubleshooting guides to support more confident clinical evaluations and cycler operation.

With Liberty Select 2.9, we've made it easier than ever for you to team up with patients and personalize cycler treatment settings to meet their individual needs and goals. To learn more about the features and benefits of Liberty Select, and how to access our new training tools, contact your Fresenius Medical Care Home Therapy Clinical Team.

INDICATIONS FOR USE: The Liberty Select cycler is indicated for acute and chronic peritoneal dialysis. The stay•safe PIN connectors are intended for use with a peritoneal dialysis (PD) cycler for drainage and infusion of PD solution during PD exchanges. The stay•safe PIN connectors are indicated for use in acute and chronic peritoneal dialysis.

CAUTION: Federal (US) law restricts these devices to sale by or on order of a physician.

NOTE: Read the Instructions for Use for safe and proper use of these devices. For a complete description of hazards, contraindications, side effects and precautions, see full package labeling at www.fmcna.com.

