## Liberty<sup>®</sup> Select Patient Skills Checklist

For software versions 2.8.8 and higher



	$\sqrt{1}$ Indicates	Indicates Skill Performed Correctly			
Skill Demonstrated	Simulated	1st Tx	2nd Tx		
15. Troubleshoots alarms as instructed					
16. Identifies location of Technical Support number, software version, and serial number on cycler					
17. Writes down the alarm code prior to calling Technical Support					
18. Describes the function of the <b>Details</b> button					
19. Demonstrates how to insert the IQdrive into the cycler					
20.Progresses appropriately after a power failure					
stay•safe connection and disconnection procedure for APD (P/N 100291-01)					
1. Identifies stay•safe PIN connector					
2. Demonstrates proper procedure when connecting to cycler set					
3. Demonstrates proper procedure when disconnecting from cycler set					
4. Demonstrates proper placement of first and second PIN connector when using cycler set with dual PIN connector					

Date Skills Completed:

Patient Signature:

Nurse Signature: \_\_\_

Please refer to the Liberty Select Cycler User's Guide for detailed information on device description, instructions, contraindications, warnings, and precautions.

The Liberty Select cycler is intended for acute and chronic peritoneal dialysis. The stay-safe PIN connectors are intended for use with a peritoneal cycler for drainage and infusion of PD solution during peritoneal dialysis exchanges. The stay-safe PIN connectors are indicated for acute and chronic peritoneal dialysis.

Caution: Federal (US) law restricts these devices to sale by or on the order of a physician.

Note: Read the Instructions for Use for safe and proper use of these devices. For a complete description of hazards, contraindications, side effects and precautions, see full package labeling at www.fmcna.com.



Fresenius Renal Technologies A Division of Fresenius Medical Care North America 920 Winter Street, Waltham, MA 02451 800-662-1237 ext. HOME (4663) | www.fmcna.com

© 2019, Fresenius Medical Care. All Rights Reserved. Fresenius Medical Care, the triangle logo, Liberty, Liberty, Select, IQdrive, and stay•safe are trademarks of Fresenius Medical Care Holdings, Inc. or its affiliated companies. All other trademarks are the property of their respective owners. P/N 103419-01 Rev A 09/2019

atient Name:		Date:			
Patient Name: Date: Clinic:					
Nurse Name:					
Skill Demonstrated	$\sqrt{1}$ Indicates S	Indicates Skill Performed Correctly			
	Simulated	1st Tx	2nd Tx		
Setting up the Liberty Select Cycler for Treatment (Tx)					
<ol> <li>Locates white power switch on back of machine and sets it to On position</li> </ol>					
2. Checks treatment settings on the "Ready" screen					
<ol> <li>Confirms treatment settings are correct and presses</li> <li>OK to start treatment</li> </ol>					
<ol> <li>Follows the step-by-step on-screen instructions exactly. Presses Next when indicated on each screen</li> </ol>					
5. Gathers correct supplies for the treatment per clinic instructions					
<ol><li>Puts on mask and ensures that everyone else in the room puts on a mask</li></ol>					
7. Cleans hands thoroughly per clinic procedure					
<ol> <li>Opens the cassette package with the tubing and removes it from the package. Places the patient line in the stay•safe<sup>®</sup> organizer.</li> </ol>					
9. Verifies that Liberty cassette is free of kinks and damage.					
10. Locates and opens cassette door					
11. Snaps the cassette in place correctly. Does not touch the door sensor when loading the cassette. Closes the door.					
12. Connects the lines to solution bags when prompted to by the cycler					
13. Places a solution bag on the heater tray in correct position. Follows screen instructions.					
14. Breaks the cones when instructed to do so by the cycler. Clamps any unused lines.					
15. Verifies the bags are connected per screen indicator					
16. Verifies that only one heater bag is left on the heater tray					

## Liberty<sup>®</sup> Select Patient Skills Checklist



## Liberty<sup>®</sup> Select Patient Skills Checklist

For software versions 2.8.8 and higher

Chill Domonotootod	$\sqrt{1}$ Indicates Skill Performed Correct		$\checkmark$ Indicates Skill Performed Correctly				
Skill Demonstrated	Simulated	1st Tx	2nd Tx			Skill Demonstrated	
17. Ensures blue clamp of patient line is open after flush is complete						12. Reviews and explains the treatment sum the nurse	
<ol> <li>Ensures the patient line is fully primed. Primes a second time if integrated set is used.</li> </ol>						13. Presses <b>Next</b> to continue	
-						14. Turns off the cycler	
<ul><li>19. Closes patient line (blue clamp)</li><li>20. Connects the catheter extension set to the PIN connector.</li></ul>				-		15. Cleans the cycler with (1:100) bleach solu off with a water-dampened cloth	
Opens patient line (blue clamp) and catheter extension set (white clamp). Removes PIN connector from the stay•safe organizer.						<b>General Machine and Disposables Knowledg</b> 1. Identifies heater/scale, control panel, door	
21. Presses Next to start treatment				_		On/Off switch	
Discontinuing the Liberty Select Cycler Treatment						2. Demonstrates how to handle alarms by p to mute, identifying alarm code, and trou	
<ol> <li>Follows the step-by-step on-screen instructions exactly. Presses Next when indicated on each screen.</li> </ol>						as taught 3. Can name the four hard key buttons	
2. Turns the blue button on the PIN connector clockwise to unlock, then pushes the blue button to insert the PIN into the catheter extension set. Closes blue clamp on patient					$\bigcirc$	<ul><li>4. States the colors of the status bar and wl color indicates</li></ul>	
line and white clamp on catheter extension set.						5. Demonstrates how to check "Treatment S	
<ol> <li>Places a new stay•safe cap in the left side of the stay•safe organizer</li> </ol>						6. Demonstrates how to check "My Records	
<ol> <li>Inserts the PIN connector that is connected to catheter extension set into the stay•safe clip of the organizer</li> </ol>						7. Demonstrates how to set "Options" for cy including SI units	
<ol> <li>Removes the protective cover from the new stay•safe cap and sets it aside</li> </ol>						8. Identifies the color of the cassette line cla each color indicates	
6. Unscrews the catheter extension set from the						9. Identifies PIN connector on cycler set and blue button and PIN	
PIN connector				 		10. If able, lifts the cycler properly when mov	
<ol> <li>Connects the catheter extension set to the new stay•safe cap</li> </ol>						<ol> <li>Demonstrates how to accept or reject the the IQdrive<sup>™</sup></li> </ol>	
8. Removes the catheter extension set from the organizer notch. Makes sure the cap is tight.						12. Demonstrates how to change treatment p "My Settings" screen	
9. Presses the <b>Next</b> button on the cycler						13. Presses <b>Ok</b> to save changed settings prio	
10. Closes all clamps on the cycler set. Removes the cassette from the cycler and closes the door.						leaving a screen when a change has been during programming	
11. Presses <b>Next</b> to continue						14. Differentiates between connected and un in the "Verify Bags" screen	



onstrated	$\sqrt{1}$ Indicates Skill Performed Correctly					
onstrated	Simulated	1st Tx	2nd Tx			
eatment summary to						
)) bleach solution and rinses it cloth						
les Knowledge						
rol panel, door, and						
e alarms by pressing <b>Stop</b> ode, and troubleshooting						
buttons						
us bar and what each						
"Treatment Settings"						
"My Records"						
options" for cycler operation,						
assette line clamps and what						
cycler set and can point to the						
erly when moving it						
t or reject the settings from						
e treatment parameters on the						
l settings prior to Inge has been made						
nected and unconnected lines						