stay-safe® Exchange Set CAPD System Peritoneal Dialysis In-service Checklist



Name: Date:	
Facility:	
Clinical Support Specialist:	
Training Topics-	
Topics may be modified to accommodate the specific policies and procedures of the unit being trained. Check	s indicate
topics reviewed during in-service training; no representation is made regarding participant competence or expe	rtise.
stay • safe® Exchange Set CAPD System—Perform or Describe the Following:	
1. Prepares work area	
a. Closes windows, shuts off fans, removes pets	
b. Cleans work surface with appropriate disinfectant (per facility policy and procedure)	
c. Masks	
d. Washes and dries hands (per facility policy and procedure)	
2. Gathers supplies	
a. Warmed stay • safe CAPD solution bag	
b. stay•safe cap	
c. stay•safe organizer	
d. stay•safe Handi-tool	
e. IV pole	
f. Solution bag scale	
g. Mask(s)	
h. Hand sanitizer	
i . CAPD treatment log and pen or pencil	
3. Prepares stay•safe system	
 a. Secures stay • safe organizer to IV pole with holders, or press suction cups onto flat surface at an appropriate height (in front of seated patient) 	
b. Warms stay • safe solution bag according to manufacturer's instructions	
 Removes solution bag from overwrap, by tearing from a notched edge of the package recognizing that some bag opacity and condensation is normal 	3
d. Places the set on the work surface. Separate the fill and drain bag.	
e. Checks solution bag according to S.C.A.L.E.*	
f. Removes disc cover by turning dial counterclockwise to fit within cut-out	
g. Checks that stay•safe dial points to Position 1	
h. Places stay•safe disc and tubing into organizer	
i . Hangs solution bag from IV pole	
j . Breaks frangible (cone) in solution bag (for assistance use Handi-tool)	60
k. Places drain bag on floor (shiny side up)	
I. Opens stay • safe cap package and inserts covered cap into one of the notches on the front of organizer	

^{*} Solution, Clarity, Amount, Leaks, Expiration Date

stay-safe® Exchange Set CAPD System Peritoneal Dialysis In-service Checklist



stay • safe® Exchange Set CAPD System—Perform or Describe the Following:	
4. Connects	
a. Sanitizes hands (per facility policy and procedure)	
b. Makes extension set accessible	
c. Inspects extension set verifying that clamp is closed	
d. Inserts extension set cap into the other organizer notch	
e. Removes cap from disc (patient connector) in organizer	
f. Twists extension set to remove cap	
g. Immediately connects extension set to disc	
h. Removes mask. The system will not be open again during the exchange.	
5. Opens the extension set clamp. Patient will start draining	
6. When drain is complete, turns dial to Position 2, starting a 5 second flush	
7. After 5 seconds, turns dial to Position 3 starting the patient fill	
8. When fill is complete, turn the dial to Position 4. This will insert the closure PIN into the extension set connector. You will feel a "click" when you get to the last position.	
9. The system is now sealed.	
10. Disconnects	
a. Closes clamp on extension set	
b. Removes the white protective cover from the new stay•safe cap and reserves	
for later use. Do not discard.	
c. Removes extension set from stay • safe disc and attach the new stay • safe cap. Twist clockwise to secure the connection. (notes PIN in extension set)	
d. Inserts extension set into uncovered stay•safe cap in organizer (covering PIN)	
e. Removes covered extension set from organizer	
f. Secures extension set	
g. Weighs drain bag and records weight	
11. Cleans up	
a. Covers cap left in organizer with cover from new cap	
b. Removes stay•safe disposable system from organizer	
c. Discards solution and disposables (per facility policy and procedure)	
d. Cleans stay • safe organizer (per facility policy and procedure)	

This item is not intended to replace the Instructions for Use (IFU) found in each box of DELFLEX® Dextrose Peritoneal Dialysis Solutions with attached stay • safe exchange set.

Indications for Use: DELFLEX® peritoneal dialysis solutions are indicated in the treatment of chronic renal failure patients being maintained on continuous ambulatory peritoneal dialysis when nondialytic medical therapy is judged to be inadequate.

Caution: Federal (US) law restricts this device to sale by or on the order of a physician or other licensed practitioner. Read the Instructions for Use for safe and proper use of this device. For a complete description of hazards, contraindications, side effects and precautions, see full package labeling at www.fmcna.com.

