

# New Features.


Reduced alarms and more personalized therapy options with **Liberty<sup>®</sup> Select 2.9**




Welcome to your Liberty Select 2.9, the latest update to the Liberty Select cyclor! Please read and reference this flyer and the additional training tools referenced on the back to familiarize yourself with the new features. For medical or clinical questions about these and any other features, please call your home therapy nurse or nurse on call.

## Fewer Sounds, Sounder Sleep


### Optional Flow Alert Notification

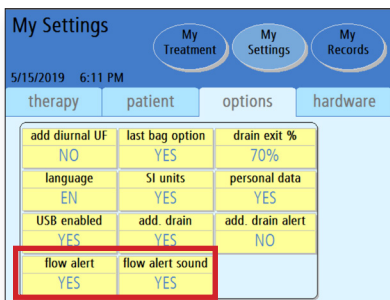

*Default Option:* Flow Alert and Flow Alert sound are ON.

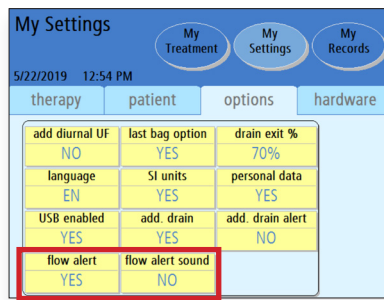


*Option 2:* You can choose to receive an on-screen alert without an audible alarm.

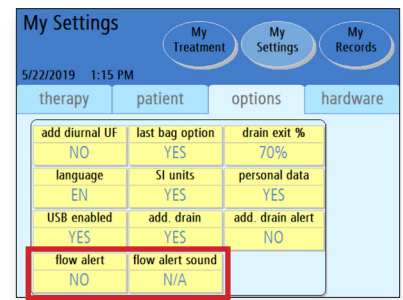


*Option 3:* You can choose to turn both Flow Alert notifications OFF.

## Less Cause for Alarms


### Removal of the Scale

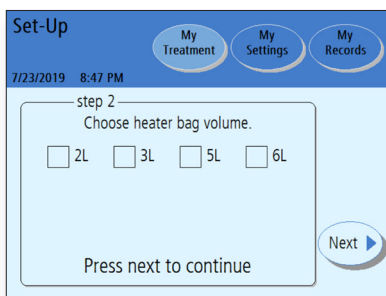

Choose your heater bag volume during initial setup.

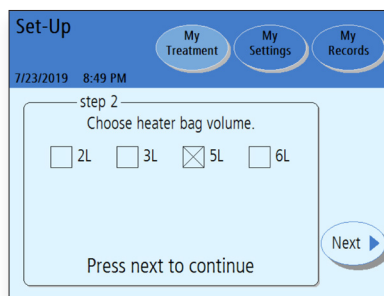

Once selected, your choice becomes the default setting for your future treatments. You won't have to change it again unless your bag volume changes.

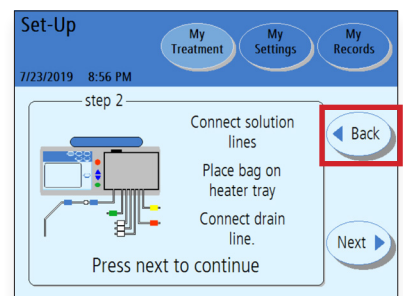

If needed, press **Back** to change your heater bag volume.

See reverse for more information.

# New Features.

Reduced alarms and more personalized therapy options with **Liberty<sup>®</sup> Select 2.9**



## A More Personal Touch Easier Data Entry and Monitoring

Patient Data

7/26/2019 9:55 AM

Your personal health data  
Enter your health data below

dextrose 1.5%	last bag type	pulse
dextrose 2.5%	blood glucose	blood pressure
dextrose 4.25%	weight	temperature

Press next to continue

Value boxes are blank when the Personal Health Data screen first appears. This screen only appears after a completed treatment if the *personal data* option is set to **YES**.

Patient Data

7/26/2019 9:54 AM

Your personal health data  
Enter your health data below

dextrose 1.5%	last bag type	pulse
500 mL	1.5	60
dextrose 2.5%	blood glucose	blood pressure
500 mL	20 mg/dL	120 / 70
dextrose 4.25%	weight	temperature
1000 mL	105 lbs	98.2 °F

Press next to continue

Touch each individual box. When the color changes and the previous treatment values appear, use the Up/Down arrows to change the value.

## For More Information:

**Flow Alert:** *Liberty Select Handi-Guide*, page 19;  
*Liberty Select User's Guide*, pages 91-92

**Heater Bag Volume:** *Liberty Select Procedure Card*, step 10;  
*Liberty Select Handi-Guide*, page 9;  
*Liberty Select User's Guide*, pages 110-111

**Data Entry & Monitoring:** *Liberty Select Handi-Guide*, page 32;  
*Liberty Select User's Guide*, pages 158-160

**INDICATIONS FOR USE:** The Fresenius Liberty Select cyclor is indicated for acute and chronic peritoneal dialysis.

**CAUTION:** Federal (US) law restricts these devices to sale by or on order of a physician.

**NOTE:** Read the Instructions for Use for safe and proper use of these devices. For a complete description of hazards, contraindications, side effects and precautions, see full package labeling at [www.fmcna.com](http://www.fmcna.com)

