


Checking Your Solution Bags for Moisture and Leaks

PD

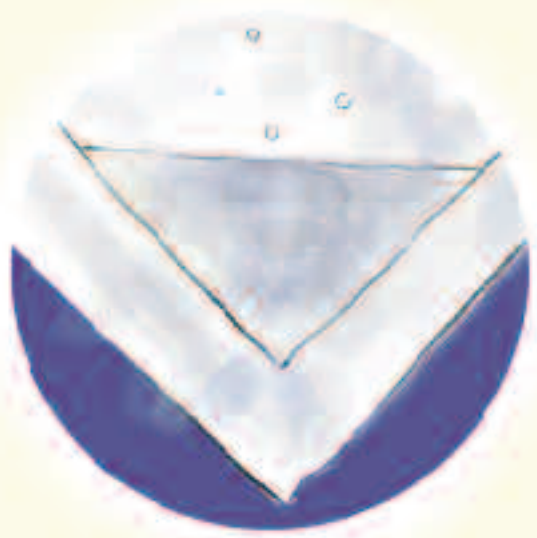

This amount of moisture is normal

Follow these instructions to check your solution bags for leaks:

- Inspect the bag before opening the outer wrapping. If the outer wrapper contains an excessive amount of fluid, do not use the bag.
- If the bag does not contain an excess amount of fluid, remove it from the outer wrapper and using a clean, dry towel carefully wipe any moisture from the surface or edges of the bag
- Place the bag flat on a clean, dry surface
- Thoroughly inspect the bag for leaks while squeezing it firmly
- Flip the bag over and repeat the above steps for the other side.
- If a leak is detected, do not use the bag and contact Fresenius Medical Care Customer Service at 800-323-5188.

Please keep the sample until after you have contacted Customer Service!



