



Fresenius Medical Care

Fresenius Medical Care North America Service Bulletin

Equipment: 2008K² Hemodialysis Machines
Bulletin: 10-FK2-001 Rev A
Subject: Functional Software Upgrade to 5.22 or Greater

Originated By (Dept.):	Reviewed By	Approved By	Effective Date
J. Fox (TSD)	Kenneth Cogswell	ECO# 31556	04/07/2010

Distribution: Dir TS (B. Rogers), TS Mgr (K. Cogswell), Dir QS (M. Owen), QE (R. Fox), Eng (D. Gavin), QC Mgr (D. McKenzie), Tng (J. Barr), Prod Mgr (R. Few), Sean McFaul (Canada), Jorge Naranjo (Mexico)

1.0 PURPOSE

To inform all Field Service and Technical Support personnel of a mandatory Blood Pump Serial Communication cable (P/N 670658-Q) installation when upgrading a 2008K² to Functional software v5.22 or greater.

2.0 PROCEDURE

Before upgrading a 2008K² to Functional software v5.22 or greater, inspect the Arterial Blood pump for the presence of the Blood Pump Serial Communication cable. This cable is connected between the Serial Interface connector on the Arterial Blood pump main board (LP955) and P10 of the Functional board.

If this cable is not present and the 2008K² is upgraded, the alarm message "Art. BP no comm" will be displayed and the Arterial Blood pump will not run.

If there are any questions regarding this bulletin, contact Technical Support at 800-227-2572 x7003

© 2010, Fresenius Medical Care North America, All rights reserved.