



Fresenius Medical Care

Fresenius Medical Care North America Service Bulletin

Equipment: 2008T Hemodialysis Machines
Bulletin: 12-FRT-002 Rev A
Subject: Card Cage Repair Safety

Originated By (Dept.):	Reviewed By	Approved By	Effective Date
J. Fox	Brent La Bier	CN042400	05/18/12

1.0 PURPOSE


To inform all Field Service and Technical Support personnel of the recommendation for an additional person to assist when making repairs inside the card cage of the 2008T hemodialysis machine.

2.0 SAFETY INSTRUCTIONS

Caution: Care must be taken when removing the traverse rod screws on the 2008T hemodialysis machine (illustrated below).

If the front panel is not properly supported, the front panel hinges could disconnect from the card cage chassis.

The assistance of a second person to support the front panel when making repairs inside the card cage on the 2008T hemodialysis machine is recommended.

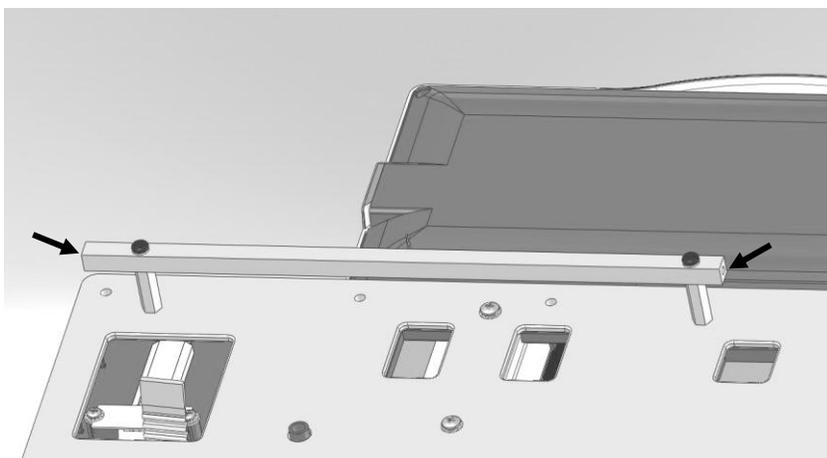

If there are any questions regarding this bulletin, contact Fresenius Technical Support at 800-227-2572.

© 2012, Fresenius Medical Care North America, All rights reserved.