



## Fresenius Medical Care North America Service Bulletin

Equipment: All 2008<sup>®</sup> Series Hemodialysis Machines  
 Bulletin: 21-FRS-002 Rev A  
 Subject: Sensor Board Compatibility

### 1.0 PURPOSE

To inform all Field Service and Technical Support personnel as well as Fresenius Medical Care-trained customers of Sensor board compatibility in the 2008<sup>®</sup> Series Hemodialysis machines.

### 2.0 SCOPE

Currently, there are two types of Sensor boards (part numbers 190332 & 191142) for the 2008<sup>®</sup> Series Hemodialysis machines.

Below are the required Actuator-Test board software versions for each type of machine and the appropriate Sensor board part number:

| Machine Type   | Actuator-Test Software Version | Sensor Board Part Number |
|--|--------------------------------|--------------------------|
| 2008 <sup>®</sup> K  | Any version                    | 190332 only              |
| 2008 <sup>®</sup> K <sup>2</sup>   | Any version                    | 190332 only              |
| 2008K@Home <sup>™</sup>  | Any version                    | 190332 only              |
| 2008 <sup>®</sup> T  | Versions less than 2.32        | 190332 only              |
| 2008 <sup>®</sup> T  | Versions 2.32 or greater       | 190332 or 191142         |
| 2008T BlueStar <sup>™</sup><br>Currently equipped with a 190332 Sensor board | Versions 2.32 or greater       | 190332 or 191142         |
| 2008T BlueStar <sup>™</sup><br>Currently equipped with a 191142 Sensor board | Versions 2.32 or greater       | 191142 only              |

An option is available so a 191142 Sensor board can be used in a non-2008T BlueStar<sup>™</sup> machine. The 2008T BlueStar<sup>™</sup> Standard Kit (part number 191137) will install the appropriate software versions onto the Functional and Actuator-Test boards.

If there are any questions regarding this bulletin, contact Fresenius Medical Care Technical Support at 800-227-2572.

© 2021 Fresenius Medical Care. All Rights Reserved