



## Fresenius Medical Care North America Service Bulletin

Equipment: 2008® Series Hemodialysis Machines  
 Bulletin: 24-FRS-002 Rev A  
 Subject: New Blood Pressure Cuffs

### 1.0 PURPOSE

To inform all Field Service and Technical Support personnel as well as Fresenius Medical Care-Trained customers of new Blood Pressure Cuffs for all 2008® Series Hemodialysis Machines.

### 2.0 DESCRIPTION

The following table shows the difference between the current blood pressure cuffs and the new Riester USA blood pressure cuffs:

**Note:** The new Riester USA blood pressure cuffs have only been tested on the following blood pressure modules M3600, TM-2915, TM-2917 & Suntech. See Section 3 for identification details.

**Note:** It is not recommended to use W. A. Baum cuffs and 10-foot tubing (P/N 370094) with the TM-2917 blood pressure module since this combination has not been verified.

| W. A. Baum          |                                    |
|---------------------|------------------------------------|
| Current Part Number | Blood Pressure Cuff Description    |
| 370027              | Child/Small Adult<br>Size 18-26 cm |
| 370011              | Adult<br>Size 25-35 cm             |
| 370028              | Adult Large Arm<br>Size 33-47 cm   |
| 370029              | Adult Thigh<br>Size 46-66 cm       |

| Riester USA     |                                    |
|-----------------|------------------------------------|
| New Part Number | Blood Pressure Cuff Description    |
| F40016608       | Adult Small<br>Size 17-24 cm       |
| F40016609       | Adult Medium<br>Size 24-34 cm      |
| F40016610       | Adult Large<br>Size 34-50 cm       |
| F40016611       | Adult Extra Large<br>Size 50-70 cm |

| Current Part Number | Accessories   |
|---------------------|---|
| 370094              | 10 ft Tubing Vinyl  |
| 370010              | Blood Pressure Cuff Set<br>Includes: <ul style="list-style-type: none"> <li>• Child</li> <li>• Large Adult</li> <li>• Adult Thigh</li> <li>• 10 ft Tubing, Vinyl</li> </ul> |

| New Part Number | Accessories  |
|-----------------|--|
| 390846          | 2.5 M Straight Tube  |
| 191383          | Blood Pressure Cuff Set<br>Includes: <ul style="list-style-type: none"> <li>• Adult Small</li> <li>• Adult Large</li> <li>• Adult Extra Large</li> </ul> |

© 2024 Fresenius Medical Care. All Rights Reserved.

If there are any questions regarding this bulletin, contact Fresenius Medical Care Technical Support at 800-227-2572.


### 3.0 BLOOD PRESSURE MODULE IDENTIFICATION

The installed blood pressure module can be identified from one of the debug screens.

#### Accessing Debug Screens

The Debug Screens can be accessed in both Dialysis mode and Rinse Mode by pressing and holding the **CTRL** key and then pressing the **up** and **down** arrow keys at the same time on the keyboard. Use the up and down arrow keys individually to scroll through each Debug Screen.

On Debug screen 3, locate the **COLIN** field.

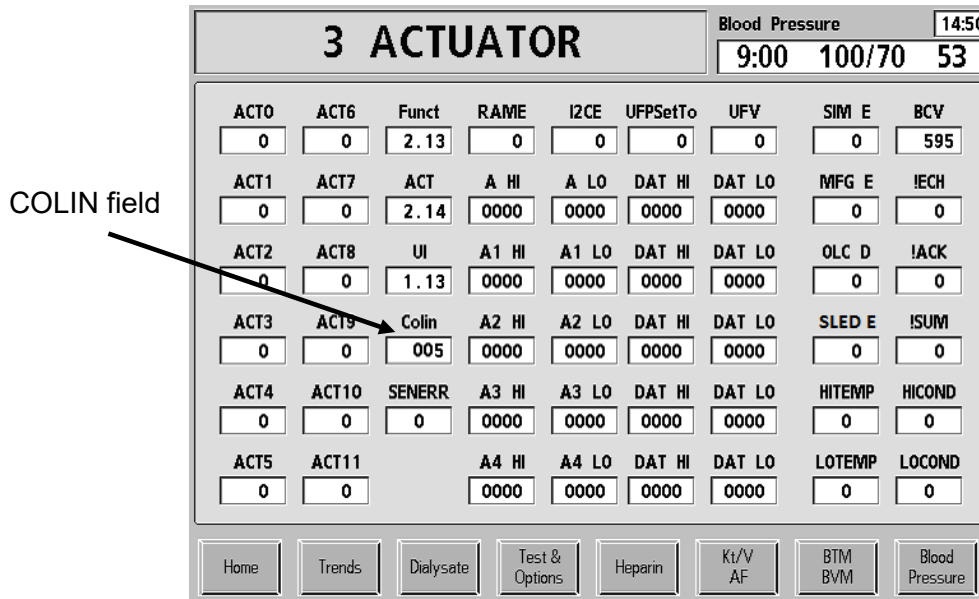

Debug Screen 3

The **COLIN** field shows specific codes based upon which blood pressure module installed:

| Blood Pressure Module | COLIN Module Code: Debug Screen 3 | Tested: Compatibility with New Riester USA Blood Pressure Cuffs Verified |
|-----------------------|-----------------------------------|--|
| M2600                 | 01                                | No   |
| <b>M3600</b>          | <b>8</b>                          | <b>Yes</b>   |
| TM-2910               | 110                               | No   |
| <b>TM-2915</b>        | <b>231</b>                        | <b>Yes</b>   |
| <b>TM-2917</b>        | <b>005 or 007</b>                 | <b>Yes</b>   |
| <b>SunTech</b>        | <b>339</b>                        | <b>Yes</b>   |

**Note:** In rare cases, the COLIN field might be blank or empty. Communication with the blood pressure module was unable to determine the module type. Power cycle the machine and try again.