

Fresenius Medical Care SERVICE BULLETIN

Equipment: Granuflo Mixer
Bulletin: 00-GRN-001
Subject: New Style Mixer Motor Coupler

Written By	Reviewed By	Approved By	Effective Date
J. Andrews	John Andrews	ECO# 7369	12/8/00


Distribution: Nat'l Svc Mgr (M. Lowery), TS Mgr (K. Cogswell), QSRA (M. Owen), QE (R. Fox), Eng (D.Gavin), DTS (M. Crnkovich), Tng (B. Hassard)

1.0 PURPOSE

To inform all Field Service and Technical Support personnel of the change to the Granuflo mixer motor shaft coupler and the elimination of white spacer used with the old style coupler.

2.0 SCOPE

The coupler that attaches the Granuflo mixer motor to the shaft has changed. There are two reasons that the coupler was changed. The first reason is that the new style coupler is longer allowing more space between the screws for a better hold. The second reason is that the new style coupler is longer eliminating the need for the white spacer. The new style coupler is the same part number as the old one (G84-001-15), but the part number for the screws has changed from G26-263 to 330045-03-01. As always you must leave 1/8" between the Granuflo mixer motor and the coupler but with the longer new style coupler you do not use the large white spacer.

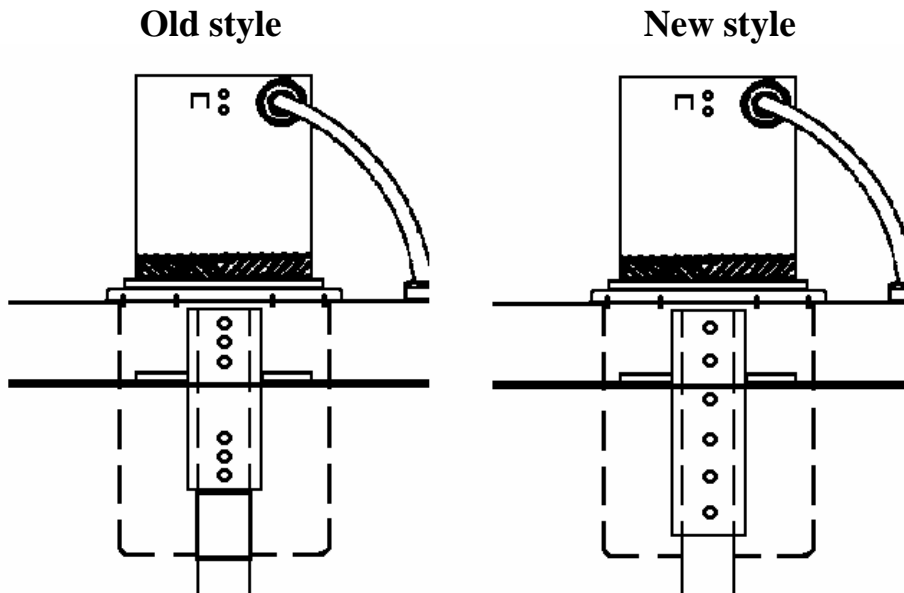

Any questions regarding this bulletin, contact Fresenius Technical Support.