

Procedure Card

Initiating Treatment


Liberty
 united with stay•safe®
 Fresenius Medical Care

This procedure card is not intended to replace the advice or training from your physician or PD Nurse. Please refer to the Liberty® Cyclor User's Guide for detailed information on device description, instructions, contraindications, warnings, and precautions. With this and all other peritoneal dialysis procedures, always maintain aseptic technique. This information is provided for quick reference only. Please follow the directions as they appear on the Liberty cyclor screens. Wait for your prompts to the next step.




1

- Make sure the cyclor is plugged into a grounded outlet
- If using the IQdrive™, insert it into the USB Port located on the back left side of the Liberty cyclor when facing the front of the cyclor
- Turn on your Liberty cyclor using the white power switch on the back of the cyclor

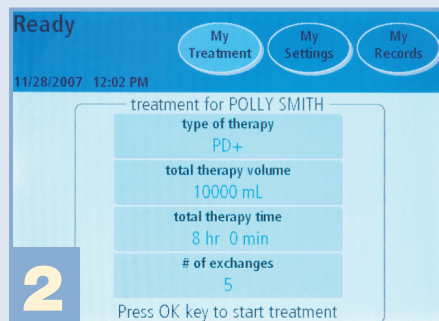

2

- To make any changes to the options or hardware choices, make them now by pressing **My Settings**
- Return to Ready screen and make sure your prescription information is correct
- Press **OK** to start treatment

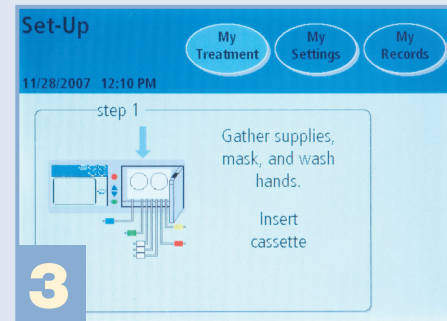

3

- Gather supplies
 - Solution bags (Check S.C.A.L.E.)
 - Liberty cyclor cassette
 - stay•safe® organizer
 - stay•safe caps
 - Drain line(s) or drain bags
 - Hand sanitizer
- Mask and wash hands



4

- Prior to insertion, inspect cassette tubing and film for possible damage
- Carefully insert the top portion of the cassette first, then press the lower portion in until it snaps securely and close the door
- Insert the blue patient line PIN connector (trigger) into the stay•safe organizer
- Press **Next**



5

When directed:

- Connect **RED** line to heater bag
- Connect **WHITE** lines to additional supply bags
- Connect **GREEN** line to last bag option
- Connect **YELLOW** line to drain line or drain bags
- Ensure heater bag is placed evenly on heater tray/scale
- Press **Next**



6


When directed:

- Break cones on the heater and solution bags with fingers or cone breaker tool
- Bend cone back and forth until completely separated
- Clamp unused lines
- Press **Next**

Procedure Card


Initiating Treatment (continued)

Liberty[®]
united with stay-safe[®]
Fresenius Medical Care

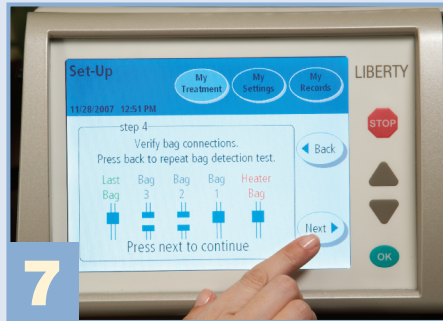

7

- Verify all solution bag connections match those on the screen
- Press **Next**
- The cyclor will now flush the solution/drain lines

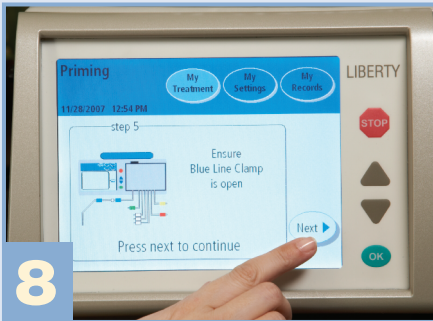


8

- Verify that the blue patient line clamp is open
- Press **Next** to prime the blue patient line




9

- When priming is complete, inspect the blue patient line to ensure it is completely filled with fluid
- Press **Back** to prime again, as necessary, or when using integrated set
- Close the blue patient line clamp and press **Next**

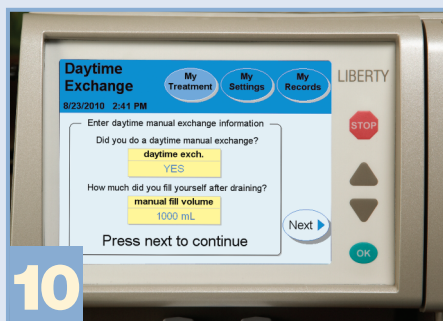

10

- Enter **YES** if you filled your abdomen with solution without using the Liberty cyclor (manual exchange)
- Enter the fill volume from your manual exchange. Press **Next** to continue.
- Review the settings just entered and then press **Next** again

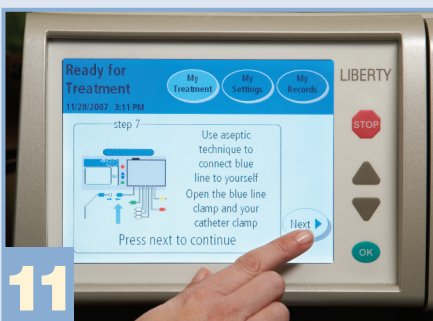

11

- Once you are connected, open both catheter and blue patient line clamps
- Press **Next** to continue
- Your treatment will now begin



12

- During Drain 0, hold the visualization chamber upside down to flush out air
- Place the visualization chamber in front of words or numbers, as directed by your clinic, to check for fluid clarity

Indications for Use: The Fresenius Medical Care Liberty cyclor is intended for acute and chronic peritoneal dialysis.

Caution: Federal (US) law restricts this device to sale by or on the order of a physician or other licensed practitioner. For safe and proper use of this device refer to the device's Instructions for Use at www.fmcna.com.



Manufactured By:
Fresenius USA, Inc.
4040 Nelson Ave
Concord, CA 94520

For all inquiries or questions, please contact technical services
1-800-227-2572

Fresenius Medical Care, the triangle logo, stay-safe, IQdrive and Liberty are trademarks of Fresenius Medical Care Holdings, Inc. and/or its affiliated companies. All other trademarks are the property of their respective owners.
©2007-2013 Fresenius Medical Care North America. All rights reserved. P/N 480070 Rev A. 12/2013