

# stay•safe<sup>®</sup> Drain Set

**1 Unit:** Single Use Disposable

**Cat. No. 026-20030** 

Compatible with stay•safe<sup>®</sup> peritoneal dialysis connectology system

## **DIRECTION FOR USE:**

### **TO CONNECT**

1. Gather the supplies, mask and wash hands.
2. Insert the Patient Connector into the blue adapter in the **stay•safe<sup>®</sup>** Organizer.
3. Close clamp on the Extension Set and insert the end into one of the notches on the Organizer.
4. Remove the protective cover from the Patient Connector on the end of the **stay•safe<sup>®</sup>** Drain Set.
5. Remove the **stay•safe<sup>®</sup>** Cap from the Extension Set.
6. Connect the Extension Set to the Patient Connector on the end of the **stay•safe<sup>®</sup>** Drain Set .
7. Remove the Patient Connector from the Organizer and open the clamp on the Extension Set to start draining.

### **TO DISCONNECT**

1. Place a new **stay•safe<sup>®</sup>** Cap in a notch on the Organizer.
2. Turn the blue button of the Patient Connector clockwise until it stops.
3. Push in the blue button of the Patient Connector until it stops.
4. Close the clamp on the Extension Set and **stay•safe<sup>®</sup>** Drain Set.
5. Insert the Patient Connector into the Organizer.
6. Remove the protective cover from the new **stay•safe<sup>®</sup>** Cap.
7. Unscrew the Extension Set from the Patient Connector.
8. Connect the Extension Set to the new **stay•safe<sup>®</sup>** Cap and remove from Organizer.

### **SAMPLING PROCESS**

As per clinical procedures.



Fresenius Medical Care North America  
Waltham, MA 02451  
1-800-323-5188  
Assembled in Mexico

### **STERILE PRODUCT:** ETO Sterilized.

Area Under Protective Guards and fluid pathway is sterile and nonpyrogenic.  
This product and package does not contain latex.

**STORAGE:** Store at room temperature.  
Protect from freezing and excess of heat.

**CAUTION:** Federal Law (U.S.) restricts this device to sale by or on the order of a physician.

### **WARNING:**

**For single patient use only. Do not reuse, reprocess or resterilize this product.  
The reuse of the line may increase the risk of peritonitis.**

This product and/or its components are protected under US patents 6,482,189 and comparable patents issued or pending in other countries.

71-4246.A