



Fresenius Medical Care

Fresenius Medical Care North America Service Bulletin

Equipment: All Peritoneal Cyclers
Bulletin: 09-PD-001 Rev B
Subject: Peritoneal Cyclers Preventive Maintenance Requirements (Update)

Written By	Reviewed By	Approved By	Effective Date
J. Fox	B. La Bier	CN042931	09/14/12

Distribution: TS Dir (B. La Bier), TS Mgr (T. Cortez, K. Cogswell), QS (M. Owen), QE (R. Fox), Eng (C. Woods), R&D (K. Plahey), Mkt (S. Flores)

Fresenius Peritoneal Cyclers:

- Freedom® Cyclers
- Freedom® Cyclers PD+
- Freedom® Cyclers PD+ IQcard™
- Liberty® Cyclers
- Newton IQ®
- 90/2® Peritoneal Cyclers

Peritoneal Cyclers Preventive Maintenance:

All Fresenius Peritoneal Cyclers are designed to detect drifting calibrations and mechanical failures through an extensive self-check built into the machine. This check is done each time the machine is powered on. Functional parameters are also continuously monitored throughout the treatment. Our history on these machines indicates that the only required maintenance is surface cleaning as stated in the specific cyclers Operator's Manual.

If problems do occur, in most cases, Fresenius will either replace or repair the cyclers within 1 working day and if applicable, make arrangements to pick up the used cyclers. The replacement policy applies only to those cyclers that remain the property of Fresenius USA, Inc. History files on these devices are maintained by Fresenius USA, Inc.

If there are any questions regarding this bulletin, contact Fresenius Technical Support at 800-227-2572.

© 2009 - 2012, Fresenius Medical Care North America, All rights reserved.