



# Fresenius Medical Care

## Fresenius Medical Care North America Service Bulletin

Equipment: 2008T Hemodialysis Machines  
 Bulletin: 12-FRT-004 Rev A  
 Subject: UI-MICS Board Touchpad Reset Board Requirement

Originated By (Dept.):	Reviewed By	Approved By	Effective Date
J. Fox (TSD)	E. Cordova	CN042706	05/22/12

Distribution: Dir TS (B. La Bier), TS Mgr (K. Cogswell), Dir QS (M. Owen), QE (M. Mohr), Eng (C. Woods), QC Mgr (D. McKenzie), Prod Mgr (G. Reny), FMS TS Mgr (J.Barr), Canada (T. Greene), Mexico (J. Naranjo)

### 1.0 PURPOSE

To inform all Field Service and Technical Support personnel of a change in the UI-MICS board requirement to have a Touchpad reset board installed.

### 2.0 SCOPE

There are two types of UI-MICS boards in the field; one that requires a Touchpad Reset board and one that has the Touchpad Reset board circuitry incorporated into the board.


The difference in the two types of boards can be identified by the orientation of Touchpad connector and its designation on the UI-MICS board.

- Boards with a Touchpad connector **J10** require a Touchpad Reset board to be installed in connector J9.
- Boards with a Touchpad connector **J10S** has the Touchpad Reset circuitry integrated into the UI-MICS board and does not require a separate Touchpad Reset board to be installed.

Below are pictures showing the different orientation of the Touchpad connector and the different connector designation:



Requires Touchpad Reset Board

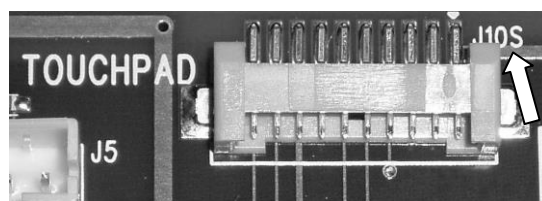

Does not require Touchpad Reset Board

If there are any questions regarding this bulletin, contact Fresenius Technical Support at 800-227-2572.

© 2012, Fresenius Medical Care North America, All rights reserved.