

# PREVENTIVE MAINTENANCE CHECKLIST SIX (6) MONTH

SERIAL NO. \_\_\_\_\_ DATE \_\_\_\_\_

MACHINE I.D. NO. \_\_\_\_\_ MACHINE HOURS \_\_\_\_\_

TECHNICIAN(S) \_\_\_\_\_ LABOR HOURS \_\_\_\_\_

See the Referenced Section of the 2008T Preventive Maintenance Procedures (part number 508033 Rev. H) for detailed instructions to perform the following:

Reference Section	Procedure	Value	Completed By:
2.1	Clean Filters.....(Replace If Necessary)		_____
2.2	Inspect Pre-UF Pump filter.....(Replace If Necessary)		_____
2.3	Inspect UF Pump Check Valves.....(Replace If Necessary)		_____
2.4	Check High Voltage AC Connections.....(<0.2 ohms)	_____	_____
2.5	Calibrate UF Pump.....(Range: 24 ± 0.1cc at 24 UF Pump strokes)	_____	_____
2.6	Check Conductivity Calibration.....(Range: $\Delta \pm 0.1\text{mS/cm}$ )	Machine Display _____ External Meter _____	_____
2.7	Check Temperature Calibration.....(Range: 37 ± 0.3°C)	_____	_____
	(Range: 39 ± 0.3°C)	_____	_____
2.8	Check Alarm Operation and Pressure Holding Test.....		_____
2.9	Verify pH.....(Range 6.9 to 7.6)	_____	_____
2.10	Check Power Failure Alarm.....		_____
2.11	Final Checks.....		_____

**NOTES:**

\_\_\_\_\_  
 \_\_\_\_\_  
 \_\_\_\_\_  
 \_\_\_\_\_  
 \_\_\_\_\_  
 \_\_\_\_\_  
 \_\_\_\_\_  
 \_\_\_\_\_  
 \_\_\_\_\_